

Malem Queen's Square Bladder Stimulator

MODEL PC9

To activate
press pad against skin or
use switch

Push button down for ON
Push again for OFF

Pull Down

Use 2 x AA
Alkaline batteries
Please observe
correct polarity
when replacing
batteries

How to use

1. place the
Stimulator on
the lower
abdomen
(above the
pubic bone)
before commencing
passing urine and
apply gentle pressure
to activate the vibration.
(If you are unable to
exert sufficient pressure to
activate, then use the on/off
button on the back.)

2. Keep the stimulator vibrating while urine is being passed
and for about one to two (1-2) minutes after you feel you have finished.

Model PC9



Adjust rope
length by tying
a knot

Use Alkaline batteries ONLY